

DON N95 & SEAL CHECK

1




Perform Hand Hygiene

2



Place mask into your palm
with metal nose piece facing up

3

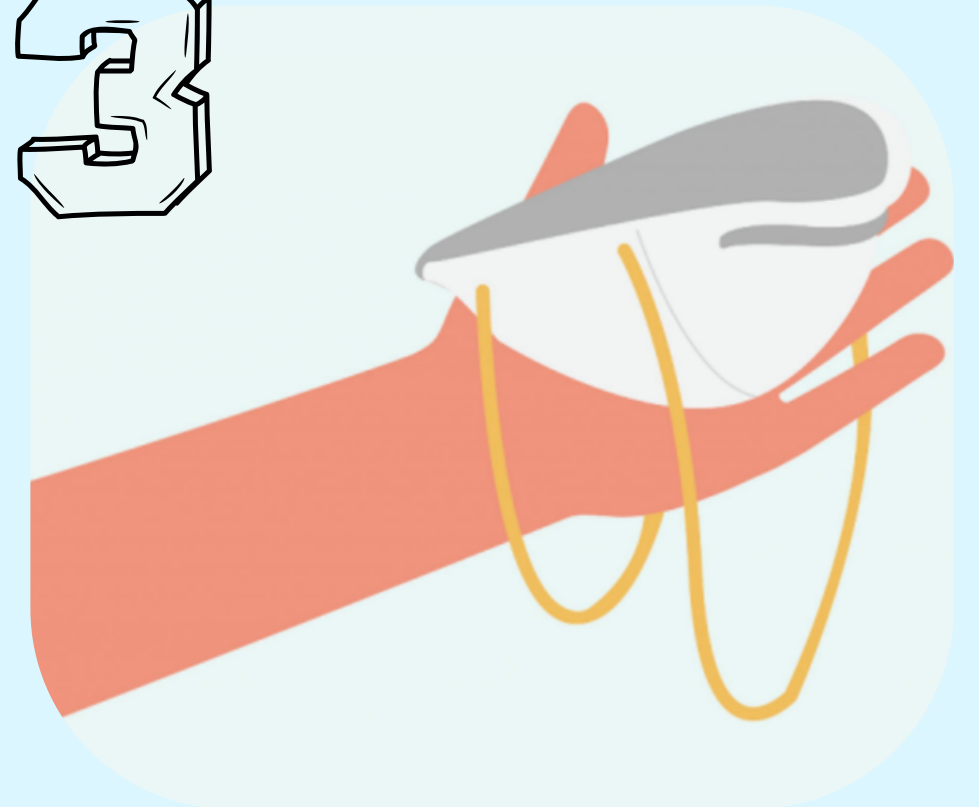

Let straps hang under your hand

4



Place mask onto your face with
metal strip across nose bridge

5



Place top strap at crown of head,
place lower strap at base of head

6



Mold nose piece for snug fit

7


How to **Seal Check** your N95

- Check for gaps by placing your hands over the N95 and exhaling
- If air leaks or glasses fog, it's not sealed—adjust and repeat



DO NOT ENTER

the isolation room if you
cannot achieve a proper seal

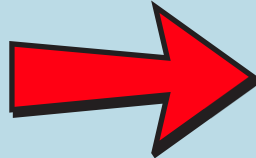For more detailed
information 
scan QR code



DOFFING A N95 RESPIRATOR



HALTON
IPAC HUB

1



- Clean your hands

2



- Bend forward and pull the bottom strap over your head, then the top strap
- Dispose in garbage
- **DO NOT** touch the front of the mask

3



- Clean your hands

For more detailed
information
scan QR code

